



DeVilbiss IntelliPAP® Standard Plus CPAP System

Enhanced comfort and reporting.

The IntelliPAP Standard Plus features DeVilbiss SmartFlex® technology, a unique exhalation pressure relief function. The IntelliPAP Standard Plus includes expanded reporting capabilities through the on-board SmartCode® Compliance Tracking and optional SmartLink® Therapy Management System.

With reliable electronics, water-resistant housing, durable Lexan® humidifier chamber and stainless steel heat transfer plate, the IntelliPAP system is built for years of dependable performance and patient satisfaction. It's just what you would expect from DeVilbiss.



DV53D Series

Performance

- **Quality** – Less than 1% failure rate and backed by an industry-leading 3-year warranty
- **Compliance** – Free remote tracking of compliance data through SmartCode Technology
- **Comfort** – Optimal humidification and SmartFlex Technology
- **Design** – Stacked design creates an exceptionally small footprint and whisper-quiet operation (26 dBA)*

DeVilbiss IntelliPAP Standard Plus CPAP System


Design

- Easy and intuitive setup
- Auto on/off** automatically starts flow of air with patient breath
- Eight-foot power cord provides greater positioning flexibility – no bulky power brick
- Reminders alert patient to replace or check parts
- Includes attractive travel bag
- Large, slip-resistant pads on bottom of unit to minimize movement on nightstand



Humidifier Chamber

- Overfill protection
- Corrosive-resistant, stainless steel plate
- Unique baffling increases air water exchange for optimal humidity
- 400 ml capacity for 8 hours of use at the highest setting

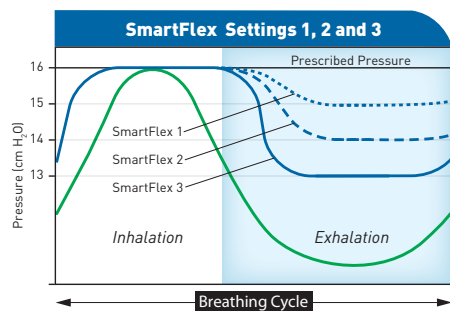

SmartFlex relief offers three pressure reduction settings that will automatically reduce the pressure generated by the IntelliPAP upon exhalation and raise the pressure back up to prescribed pressure at the onset of inspiration. The settings correlate to an exact $\text{cm H}_2\text{O}$ pressure drop. A setting of 1 = a $1 \text{ cm H}_2\text{O}$ drop in pressure during exhalation.



Usage Data	
Last 90 Days	
% Days of at least 4 hours (not continuous) in a 24 hour period	64%
Day Count	00
Days at least 4 hours	58
AHI	8.5
High Leak Flow Time	5%
NIQ	1.75
EPI	.2
SmartCode	TMTJT-4594-7DV2P
Adherence Score (4 continuous hours)	97% (20/30)

SmartCode

- Encrypted usage data codes that can be communicated by phone, e-mail or text to generate compliance and clinical reports



The Aloha™ Nasal Pillow System is the perfect fit with IntelliPAP



DeVilbiss Healthcare | 100 DeVilbiss Drive | Somerset, PA 15501 | USA
800.338.1988 | 814.443.4881 | www.DeVilbissHealthcare.com